RELEASED
Feb 20 2020

Karleen Deifel

KleerView™

EN-Cheek and Lip Retractor/DE-Wangen- und Lippenhalter/FR-Écarteur de joue et de lèvre/NL-Wang- en lipretractor /IT-Divaricatore per guance e labbra/ES-Retractor de mejillas y labios/PT-Afastador de bochechas e lábios/SV-Läpp- och munvinkelhållare/DA-Mundspærre/FI-Poskien ja huulten levittäjä/NO-Kinn- og leppeholder/EL-Παρειοκάτοχο και χειλιοκάτοχο/BG-Ретрактор за бузи и устни/HR-Obraza za usne i usne/CS-Reotraktor tváře a rtů/ET-Pöse- ja huulere- traktor/HU-Ajak- és szájterpesz/LV-Vaigu un lūpu atvirzītājs/LT-Skruostų ir lūpų retraktorius/MT-Tal-Ħaddejn u Ritrattatur tax-Xufftejn/PL-Rozwieracz policzków i warg/RO-Depărtător de obraji și buze/SK-Retraktor na pery a líca/SL-Retraktor za ličnice in ustnice/RU-Ретрактор для щёк и губ/KO-뺨 및 입술 견인기/ZH-脸颊和嘴唇牵开器/JA-チークアンドリップ・リトラクター/TH-เครื่องมือนำทางของ =พุงแก้มและปาก/TR-Yanak ve Dudak Ekartörü/AR- قش لاو دخل دا عجم

Version 1.5 - 24.10.2019

Manufactured by
Pro-Options AB, Brännkyrkagatan 94
117 26 Stockholm, Sweden

Distributed by
Ultradent Products, Inc. 505 West Ultradent Drive
(10200 South), South Jordan, Utah 84095

ULTRADENT.COM

© 2020 Ultradent Products, Inc.
All rights reserved. A1820.01 021020

EN - KleerView Cheek and Lip Retractor - User Instructions

DEVICE DESCRIPTION

Very flexible and easy to use.

INDICATIONS

To achieve working area accessibility, KleerView's transparent material is ideal for clinical photography.

- Orthodontic bonding of brackets
- Intra proximal reduction/reshaping
- Clinical photography

INSTRUCTIONS

Please see below standard operational guidance:

1. To facilitate patient comfort, apply lip balm or petroleum jelly to the patient's lips prior to inserting KleerView cheek retractors.
2. Hold both ends and squeeze the retractors together, gently bending the connecting bow in the cheek retractor.
3. The bow is positioned at the patient's chin and provides necessary tension to hold the device in place.
4. Instruct the patient to open his/her mouth. Insert the intraoral expanders of each cheek retractor against the inside of the patient's cheeks. Gently guide the patient's lips to fit comfortably in the curve of each lip retractor.
5. To remove KleerView from the mouth, carefully grasp the left and right cheek handles. Perfect for lingual bonding with excellent view for upper arch.

CLEANING REQUIREMENTS

General cleaning before sterilization: use a mild cleanser and soft brush or sponge, clean and rinse KleerView of any debris, blood or saliva before placing in a cold disinfection solution. Place elastic band around the upper/lower cheek and lip retractor to reduce tension.

1. Place in disinfectant at 90 degree Celsius program or;
2. place in single wrap in a plastic pouch and autoclave at 121 degree Celsius program
3. Dry with a lint free cloth.
4. Do not dry heat sterilize.

TRANSPORTATION AND STORAGE INSTRUCTIONS

When transporting and storing the product the following requirements should be observed:

- Store below 26°C and out of direct sunlight.
- Do not place in refrigerator.
- Plastic fatigues over time and daily use can cause breakage.
- Some chemicals and cleansers may make the retractor brittle and cloudy.
- As long as the product has not been exposed to high temperature, sunlight or humidity, it will have a long shelf life. Due to sterilization process it will become opaque, bubbles and cracks will develop. Such items should be discarded.

DISPOSAL

The device is made from recyclable polycarbonate. Dispose of according to your country or state regulations.

SYMBOLS

Class I medical device and complies with Medical Device Directive (93/42/EEC)

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