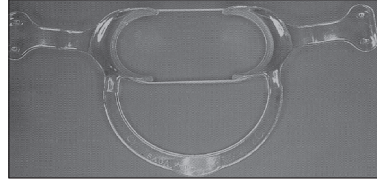
# KleerView™

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EN

Instruct the patient to open his/her mouth. Insert the intraoral extension of each cheek retractor against the inside of the patient's cheeks. Gently guide the patient's lips to fit comfortably in the curve of each lip retractor. To remove KleerView from the mouth, carefully grasp the left and right cheek handles. Perfect for lingual bonding with excellent view for upper arch.

General cleaning before sterilization: use a mild cleanser and soft brush or sponge, clean and rinse KleerView of any debris, blood or saliva before cold sterilization. Place elastic band around upper/lower cheek and lip retractor to reduce tension.

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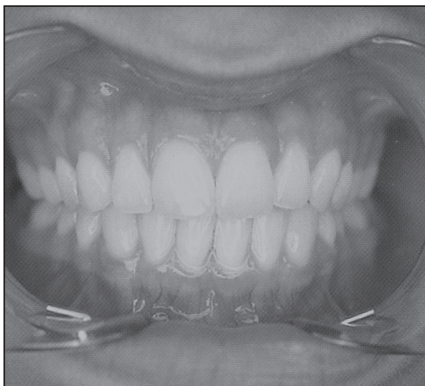
**Fig. 1**

U.S. federal law restricts this device to sale by or on the order of a dentist.

For immediate order and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call toll free: 1-800-552-5512.

Outside U.S. call (801) 572-4200, or visit [www.ultradent.com](http://www.ultradent.com).

R<sub>x</sub>

EN- Prescription use only  
 DE- Rezeptpflichtig  
 FR- Utilisation uniquement par prescription  
 NL- Uitsluitend verkrijgbaar op recept  
 IT- Soggetto a prescrizione medica  
 ES- Uso exclusivo bajo receta  
 PT- Utilização apenas por prescrição  
 SV- Receptbelagt  
 DA- Receptpligtigt  
 FI- Reseptilääke  
 EL- Μόνο με συνταγή  
 RU- Только для профессионального

EN- Consult instructions for use  
 DE- Gebrauchsanleitung beachten  
 FR- Consultez le mode d'emploi  
 NL- Raadpleeg de gebruiksaanwijzing  
 IT- Consultare le istruzioni per l'uso  
 ES- Consulte las instrucciones de uso  
 PT- Consultar as instruções de utilização  
 SV- Se bruksanvisningen  
 DA- Se brugsanvisningen  
 FI- Lue käyttöohjeet  
 EL- Συμβουλευτείτε τις οδηγίες χρήσης  
 RU- См. инструкцию

EN- Manufacturer  
 DE- Hersteller  
 FR- Fabricant  
 NL- Fabrikant  
 IT- Fabbricante  
 ES- Fabricante  
 PT- Fabricante  
 SV- Tillverkare  
 DA- Producent  
 FI- Valmistaja  
 EL- Κατασκευαστής  
 RU- Производитель

**KleeView™**

Cheek and Lip Retractor

Ultradent Products GmbH  
 Am Westhoyer Berg 30  
 51149 Cologne  
 Germany

EN- Recycle  
 DE- Recyclen  
 FR- Recycler  
 NL- Recyclen  
 IT- Riciclare  
 ES- Recicle  
 PT- Reciclar  
 SV- Återvinna  
 DA- Genbrug  
 FI- Kierrätyskesä  
 EL- Ανακυκλώστε το  
 RU- Рециркулировать

**KleeView™**

Cheek and Lip Retractor

Very flexible and easy to use!

Indications:

To achieve working area accessibility, KleeView's transparent material is ideal for clinical photography.

Instructions:

1. To facilitate patient comfort, apply lip balm or petroleum jelly to the patient's lips prior to inserting KleeView cheek retractors.
2. Hold both ends and squeeze the retractors together, gently bending the connecting bow in the cheek retractor.
3. The bow is positioned at the patient's chin and provides necessary tension to hold the device in place.
4. Instruct the patient to open his/her mouth. Insert the intraoral extension of each cheek retractor against the inside of the patient's cheeks. Gently guide the patient's lips to fit comfortably in the curve of each lip retractor.
5. To remove KleeView from the mouth, carefully grasp the left and right cheek handles. Perfect for lingual bonding with excellent view for upper arch.

Cleaning/Care

General cleaning before sterilization: use a mild cleanser and soft brush or sponge, clean and rinse KleeView of any debris, blood or saliva before placing in a cold disinfection solution. Place elastic band around upper/lower cheek and lip retractor to reduce tension.

- Store at room temperature out of direct sunlight.
- Plastic fatigues over time and daily use can cause breakage.
- Do not place in refrigerator.
- Some chemicals and cleansers may make the retractor brittle and cloudy.
- Do not dry heat sterilize or steam sterilize.

For product SDS please see our website: [www.ultradent.com](http://www.ultradent.com)  
 1-800-552-5512; 801-572-4200

**ULTRADENT**  
 PRODUCTS, INC.

© Copyright 2018 Ultradent Products, Inc. All Rights Reserved.  
 31905.09 110518