ViscoStat™ Clear 25% Aluminiumchlorid

Produktbeschreibung
ViscoStat™ Clear hemostatisk är en 25% Aluminiumklorid-gel i en viskös, aqueous vätska which leaves no residue or stain making it ideal for use in the esthetic zone. ViscoStat Clear hemostat is intended for pressure making and to control bleeding and sulcular fluids in restorative and operative dentistry used with gingival retraction cord and/or the Metal Dento-Infusor™ tips. The solutions facilitate the insertion of the cord into the sulcus.

For all products, please read carefully and understand all instructions and SDS information prior to use.

Delivery Form(s)

- 1.2 ml Delivery-syringe
- Metal Dento-Infusor™ Tip
- 30 ml IndSpense™ Spritze
- 30 ml IndSpense™ syringe

Indications for Use/Indications Purpose

Use in arrecting oral capillary bleeding and/or sulcular fluids in restorative, operative, and surgical dentistry.

Contraindications

- Do not use on patients with known allergies to aluminum chloride.
- For patients or users with allergy concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rise exposed area thoroughly with water and have patient consult their physician.

Warnings and Precautions

• Do not combine with other hemostatic agents or chemistries without first thoroughly cleaning tooth and surrounding tissue.

• Prior to bonding, impressions or in the presence of any other contaminant, the tooth/ preparation should be cleaned with a brief, 2-3 second of 35% phosphoric acid etch followed by a firm air/water spray to clean preparation. This is not necessary when using a conventional phosphoric acid etch bonding system.

• ViscoStat Clear hemostatist is designed for intraoral use.
• Patient and dental professionals must wear protective eyewear with side shields.
• Use only recommended tips.

Steps/ Instrucktioner

Preparation – 1.2 ml Syringe

1) Attach recommended delivery tip and verify flow, color, and consistency prior to applying intraorally. If resistance is met, replace tip, and recheck.

Preparation – 30 ml IndSpense™ syringe

- Attach a 1.2ml syringe to the IndSpense™ syringe.
- Express IndSpense contents into the 1.2 ml syringe to desired fill level.
- Separate syringe and re-cap IndSpense syringe.
- Securely attach recommended delivery tip.
- Verify flow, color, and consistency prior to expressing intraorally. If resistance is met, replace tip, and recheck.

Use - Impression, Direct Bonded Restoration, Indirect Bonded/Luted Restoration

- Slowly express solution while rubbing firmly against tooth.
- When bleeding or oozing ceases, use air/water spray to clean preparation. If bleeding occurs, repeat use with firm air/water spray, repeat step 1 until bleeding is controlled.
- If tissue displacement is needed continue to step 4. If displacement is not needed, cleanse tooth with 35% phosphoric acid for 2-3 seconds followed by a firm air/water spray and clean preparation with impression, direct bonded restoration, or bonding/light curing material.
- Displace tissue by packing appropriate size of UltraRac™ cord into sulcus. For optimum displacement and hemostasis, place a small amount of ViscoStat Clear hemostatist in a dispenser dish and soak cord prior to packing.
- Recheck and recheck UltraDent aluminum chloride hemostatic agents soaked in cord is around 5-10 minutes.
- Remove cord and cleanse tooth with 35% phosphoric acid for 2-3 seconds followed by a firm air/water spray and continue with impression, direct bonded restoration, or bonding/light curing material.

NOTE: For direct restoration cord may be removed or left in place to protect soft tissue. The cloth cleaning step can be performed while cord is in place.

- Apply bonding agent and restorative as material per manufacturer's instructions.

1) The tooth and surrounding tissue should be thoroughly cleaned, and all residual hemostatic agent and coagulum removed.

Processing

To avoid cross-contamination, wipe syringe with an intermediate level disinfectant between uses and use a disposable barrier sleeve. If a barrier sleeve is not used, syringe should be cleaned with an appropriate disinfectant. A clean syringe should be used for each patient.

Storage and Disposal

Dispose of waste according to local rules, guidelines, and regulations.

Miscellaneous Information

Report any serious incident to the manufacturer and the competent authority. www.ultradent.com / 1.800.552.5512 / 801.572.4200

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