UltraPro® Tx Air Hygiene Handpiece

EN

Description:

UltraPro® Tx Air Hygiene Handpiece is a light-weight, ergonomic, friction grip handpiece. The motor has a working output of 4000 RPM at variable working pressures of 35-45psi.

Indications for Use:

UltraPro® Tx handpieces are intended for prophylaxis procedures, for use by a trained professional in dentistry.

Instructions:

1. Prior to attaching air supply, bleed air line of moisture and contamination.
2. Attach handpiece (motor and nosecone) to air hose.
3. Confirm working pressure is between 35-45psi.
4. Attach prophyl angle by pushing it until it is fully seated. Friction grip will hold the angle in place.
5. To remove the prophyl angle, grasp the nose cone and firmly pull the prophyl angle in a straight outward direction.
6. For a smoother swivel between the motor and nose cone, the "O" ring may be removed from the motor stem.

Sterilization and Maintenance of Motor and Nosecone:

CAUTION: DO NOT DRY HEAT STERILIZE.

1. Remove prophyl angle from the handpiece.
2. Disconnect handpiece from hose.
3. Separate the motor section and the nosecone.
4. Clean all external surfaces with a toothbrush, cloth, or gauze moistened only with water or a neutral pH cleaning solution. Do not spray, wipe, or immerse handpiece in surgical milk, alcohol, or disinfectants.
5. Place in sterilization pouch and seal.
6. Autoclave at a maximum of 135°C (275°F) for 30 minutes.*
7. Allow to dry and cool in sterilization pouch.
8. After sterilizing, place two drops of handpiece oil into the motor air tube (Fig. 1) and 1-2 drops of oil into both ends of the nosecone (Fig. 2). To properly dispense the oil, run the motor for several seconds.
9. Apply one drop of handpiece oil to the "O" ring one time per week.

Recommended Sterilization Cycle *		
PROCESS TYPE	MINIMUM TIME in minutes at STERILIZATION TEMPERATURE	
	INSTRUMENTS:	
	WRAPPED	UNWRAPPED
Saturated Steam Standard Chamber 135°C	30	30

* or other steam autoclave conditions as validated in your facility. Do not dry-heat sterilize. Do not exceed 135°C.

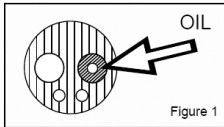
Figure 1
Lubricating Motor

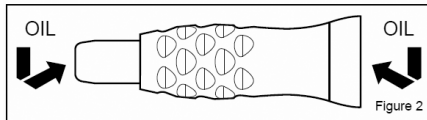
Figure 2
Lubricating Nosecone

Precautions and Warnings:

1. Carefully read and understand all instructions before using.
2. Never place handpiece in ultrasonic cleaner.
3. Do not use sterilization bags containing disinfectants.
4. Always check that the compressed air is dry and free of oil.
5. Do not exceed 45psi when operating handpiece.
6. For best results, use prophyl angles with 2.30mm – 2.35mm shanks.
7. The CDC recommends sterilization between uses.
8. Do not spray or immerse handpiece in alcohol or disinfectants.
9. Always check that the angle is firmly seated in place on the handpiece before beginning the procedure.
10. For use by individuals who are professional trained in coronal polishing.

Warranty:

Ultradent Products, Inc.'s ("Ultradent") UltraPro® Tx Air hygiene handpiece warranty obligations are limited to the terms set forth below and are not transferable: Ultradent warrants this UltraPro® Tx Air handpiece, including the motor and nosecone, against defects in materials and workmanship under normal use and proper maintenance for a period of ONE (1) YEAR from the date of purchase by the first retail purchaser ("Warranty Period"), as evidenced by invoice and shipping records. If a defect arises and a valid claim is received within the Warranty Period, Ultradent will, at its option repair or replace the defective product. The new Warranty Period for the repaired or replaced product shall be either ninety (90) days from the date of replacement or repair of the defective product or the amount of time remaining on the original Warranty Period, whichever period of time is greater. When a refund is given, the product for which the refund is provided must be returned to Ultradent. If handpiece is modified, misused or disassembled, warranty is void.

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© Copyright 2018 Ultradent Products, Inc. All Rights Reserved.