OpalDam™ Green / OpalDam™
Gingival Resin Barrier

EN

Product Description

OpalDam™ and OpalDam™ Green are unique, light-reflective, passively adhesive (sealing) light cured resins. Their methacrylate base is designed with adequate strength to maintain a barrier yet removes easily from embrasures and undercuts. The contrasting green color of OpalDam Green shows its location on the gingiva and teeth for more effective coverage.

For all products described, carefully read and understand all instructions and SDS information prior to use.

Delivery Form(s)

1.2mL syringe

Delivery tip:

- Micro 20 ga Tip
- Black Mini™ Tip

Indications for Use/Intended Purpose

OpalDam and OpalDam Green are unique, light-reflective, passively adhesive (sealing) light cured resins. Their methacrylate base is designed with adequate strength to maintain a barrier yet removes easily from embrasures and undercuts. The contrasting green color of OpalDam Green shows its location on the gingiva and teeth for more effective coverage.

Contraindications

- Do not use OpalDam/OpalDam Green resin barrier on patients with known sensitivity to methacrylates or other resins.
- For patients or users with allergy concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.

Warnings and Precautions

- Resins can be sensitizing. If dermatitis, rash, or any other allergic reaction develops wash area with lots of water and refer patient to their physician.
- Take caution when using OpalDam resin barrier for isolation on patients with severe mucositis or subepidermal autoimmune disease that manifest in the oral cavity. Instead use a rubber dam in place of resin barrier.
- If bubbles occur at the gingival margin of the OpalDam/OpalDam Green resin barrier, reapply in that area.
- To avoid cross-contamination, wipe syringe with an intermediate level disinfectant between uses

and use a disposable barrier sleeve. If a barrier sleeve is not used, syringe should be treated as single use.

- Tips are single use only.
- When curing resins on soft tissues, use scanning motion to prevent heat irritation from curing light.

Stepwise Instructions

Preparation

1. Remove luer lock cap and attach recommended tip securely onto OpalDam/OpalDam Green resin barrier syringe.
2. Verify flow prior to expressing intraorally. If resistance is met, replace tip and re-check.
3. Completely rinse and dry teeth and gingiva where OpalDam/OpalDam Green resin barrier will be applied.

Use

1. Express OpalDam/OpalDam Green resin barrier approximately 5–10 mm in height and lap approximately 2–3 mm onto enamel. Extend resin beyond the last tooth to be treated and onto open embrasures to protect interdental papilla and lingual gingival tissue. Resin buttons extending through gingival embrasures add retention to OpalDam/OpalDam Green resin barrier.
2. When protecting a full arch, light cure the resin barrier on standard mode using a scanning motion for 20 seconds and keeping the lens approximately 1 cm from the barrier and tissue. With a dental instrument, carefully check resin for a full cure. If not using on full arch, scan the barrier for approximately 3–5 seconds per tooth.

Cleanup

1. After treatment, lift OpalDam/OpalDam Green resin barrier from surface by gently sliding a dental instrument between the dam and tissue. All or large pieces will lift off at one time.
2. Remove OpalDam/OpalDam Green resin barrier interproximally using explorer and/or floss. Rinse mouth thoroughly.

Processing

To avoid cross-contamination, wipe syringe with an intermediate level disinfectant between uses and use a disposable barrier sleeve. If a barrier sleeve is not used, syringe should be treated as single use.

Storage and Disposal

Store according to outer package label. Dispose of waste according to local rules, guidelines, and regulations.

Miscellaneous Information

Report any serious incident to the manufacturer and the competent authority. www.ultradent.com / 1.800.552.5512 / 801.572.4200

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RxOnly

EN - For professional use only
ES - Solo para uso profesional

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512. Outside U.S. call (801) 572-4200 or visit www.ultradent.com.

U.S. federal law restricts the sale of this device by or on the order of a dentist.

OpalDam™ Green / OpalDam™
Gingival Resin Barrier

Manufactured by:

Ultradent Products Inc.
505 West Ultradent Drive (10200 South)
South Jordan, Utah 84095 USA
Made in the USA from U.S. and globally sourced materials

ULTRADENT
PRODUCTS, INC.

© 2023 Ultradent Products, Inc. All Rights Reserved.

For product SDS please see our website: www.ultradent.com
1-800-552-5512; 801-572-4200

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