UltraEZ™ <div>Desensitizing Gel</div>	EN
---------------------------------------	----

Product Description

UltraEZ™ desensitizing gel is a unique, sustained release, 3% potassium nitrate and 0.11% weight by weight fluoride ion, sticky, viscous gel. UltraEZ gel is designed to be used in a custom fabricated tray, disposable tray, or prefilled disposable tray application.

For all products described, carefully read and understand all instructions and SDS information prior to use.
Delivery Forms)

- UltraFit™ trays

Indications for Use/Intended Purpose

UltraEZ desensitizing gel provides a film-like varnish for sensitive teeth, sealing dentinal tubules of exposed dentin and other areas where sensitivity is a concern. The product is used either by a dental professional in the office or provided to the patient for home treatment of dentin sensitivity.

Contraindications

- Not intended for use by pregnant or nursing women.
- Not intended for patients who may swallow or aspirate tray.
- Not intended for use on patients under the age of 6 without consultation of their primary care doctor.
- For patients or users with allergy concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.

Warnings and Precautions

- If any problems occur, discontinue treatment and notify a dental professional.
- Always keep out of heat and/or sunlight.
- Excess material at insertion and residue upon removal should be brushed off and rinsed. DO NOT swallow.

Stepwise Instructions

Use – UltraFit Prefilled Tray

- U – Upper tray
- L – Lower tray

Remove from package.

- Position tray on teeth.
- Bite firmly then gently suck down on the tray for 2 seconds. Do not press tray against teeth as it pushes the gel against the gums.
- Remove outer tray leaving the white inner tray on the teeth.

- Repeat with the opposite arch if needed.
- Wear for 15 minutes to 1 hour per day or as directed by your clinician.

NOTE: Do not eat, drink, or use tobacco products while trays are in place. Do not use tray overnight.

Storage and Disposal

Dispose of waste according to local rules, guidelines, and regulations.

Miscellaneous Information

Report any serious incident to the manufacturer and the competent authority.

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