Consepsis® Scrub

Chlorhexidine Antibacterial Slurry

Description:
Consepsis® Scrub is a lightly flavored 2.0% (relative to liquid component) chlorhexidine gluconate disinfecting scrub.

Indications for Use:
Consepsis Scrub is used for removal of residual temporary cement prior to permanent cementation and/or for cleaning debris while disinfecting.
Directions for Use:

- For 1.2 pre-filled syringes:
 - Remove Luer Lock cap.
 - Securely attach working tip of choice to the syringe (White Mac® Tip is recommended).
 - Control the color and consistency of the product flow before applying by expressing a small amount onto a mixing pad or gauze. Discard any clear liquid that is expressed.
- Replace the tip with the Luer cap after use.
- For 30mL IndSpense syringe:
 - Remove Luer Lock cap from IndSpense syringe.
 - Attach a 1.2 syringe to the male threads of the IndSpense syringe.
 - Depress the IndSpense plunger while guiding the 1.2mL syringe plunger outward until the desired amount has been dispensed.
 - Separate syringes and re-cap IndSpense syringe with Luer cap.
 - Securely attach the working tip (White Mac® Tip is recommended) to the 1.2mL syringe.

- Direct placement instructions:
 - Express Consepsis Scrub directly to the tooth being treated or into rubber cup.**NOTE:** If tooth is not anesthetized, warm the syringe under hot tap water for approximately 10 seconds before applying.
- Proceed with the rubber cup or StarBrush at a slow speed until preparation is clean.
- Rinse and dry prepared area.

- Proceed with restorative placement taking care to avoid saliva contamination.
- Precautions and Warnings:
- For Professional use only. Use only as directed.
 - Read and understand all instructions before using.
 - Keep out of reach of children.
 - Confirm that patient has no known allergies to product ingredients.
 - Re-cap with the Luer Lock cap. To avoid cross contamination, disinfect and wipe syringe with an intermediate level disinfectant between uses or use a disposable syringe cover.
 - Discard tips and syringes in a sharps container. Contamination tips should not be re-used.
 - Verify flow of all syringes prior to applying intraligament. If resistance is met, replace tip and re-check. Use only recommended tips. To avoid cross contamination, do not re-use tips.
 - Verify the color and consistency of the product flow before applying by expressing a small amount onto a mixing pad or gauze. Discard any clear liquid that is expressed.
 - Store at room temperature.
 - Do not use after expiration date noted on syringe.
- NOTE:** For SDS and additional information about using our products, please go to www.ultradent.com

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Ultradent syringes have an expiration date stamped on the side of the syringe consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year.

U.S. federal law restricts the sale of this device by or on the order of a dentist.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512. Outside U.S. call (801) 572-4200 or visit www.ultradent.com.

Consepsis® Scrub

Chlorhexidine Antibacterial Slurry

Manufactured by:
Ultradent Products Inc.
305 West 10200 South
South Jordan, Utah 84095 USA

Ultradent Products GmbH
Am Westhove Berg 30
51149 Cologne Germany

Made in USA
For product SDS please
visit our website:
www.ultradent.com

ULTRADENT
PRODUCTS, INC.

© Copyright 2015 Ultradent Products, Inc. All Rights Reserved.

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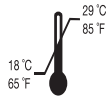
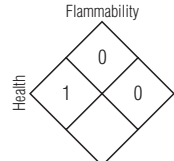
EL

Consepsis Scrub

HAZARD RATING
4 = Severe
3 = Serious
2 = Moderate
1 = Slight
0 = Minimal