veneer®

dental art made simple

EN Thank you for purchasing Uvener® templates. Please read these instructions thoroughly before use for optimal results. Uvener is a reusable template system for creating facial anatomy on anterior teeth with any composite resin.


The Uvener kit includes 32 templates:

- Two sizes (Large or Medium) both maxillary and mandibular arches.
- Each arch consists of 8 templates from centrals to premolars (bicuspid).
- The size (L or M) and the tooth number (based on the Palmer System) are embedded on the handles.

The Uvener Extra kit includes 24 templates:

- Four sizes (Extra Large, Large, Medium, and Square).
- Additional kit configurations include sets of 12 templates* (Extra Large and Square or Large and Medium).
- Each set includes maxillary teeth only from canine to canine.

* 12 template kits not available in all countries.

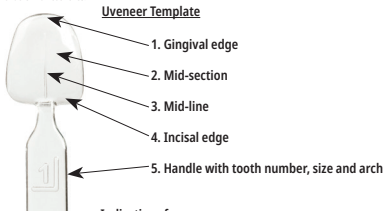Indications for use

Direct veneers, mock-ups, temporaries, and for class 3, 4, and 5 cases.

Instructions for use: Direct Composite Veneer

- Select the correct Uvener template. Ideally, the template should cover the entire facial surface of the tooth.
- Remove all caries if needed and minimally prepare the tooth.
- Place interproximal separating matrices (cellulose, Teflon, or metal matrix).
- Etch, rinse, and apply adhesive to the tooth surface and light cure according to the manufacturer's instructions.
- Apply the composite directly onto the tooth. Do not light-cure.
- Using thumb, press the concave side of the selected Uvener template onto the uncured composite on the tooth. Press firmly. In doing so, any trapped air will be removed.

- Align the centerline of the Uvener template parallel to the midline of the face and perpendicular to the incisal plane.
- Clean any excess composite from the periphery.
- Light-cure thoroughly according to manufacturer's instructions.
- Remove the Uvener template by lifting the handle and complete light curing of the composite.
- Avoiding the glossy facial surface, trim excess composite from periphery with a fine flame shaped bur and adjust incisal edge as needed with a bur or disc.
- Immediately after use, wipe templates with alcohol wipes, dry, bag, and autoclave according to product care and instructions.

Mock-Ups

Follow the same steps as direct composite veneers but do not prep, bond, or etch.

Temporary Veneers

Spot etch and spot bond and follow the same steps for direct composite veneers.

For additional information please visit www.uvener.com

Product Care and Instructions:

- Immediately after use, wipe templates with alcohol wipes in order to remove composite residue. Dry, bag, and autoclave.
- ⚠ Failure to remove composite residue after use may affect template properties including shine, and may affect the quality of the veneer, mock up, or temporary veneer.
- Use only disinfectant wipes recommended for ABS, PVC and polycarbonate (PC) plastics.
- Bag and autoclave each Uvener template at 135° C (275° F). Maximum temperature is 137° C (278° F).
- ⚠ Do not use chemical vapor sterilizers or dry heat.
- ⚠ Do not autoclave the kit base or cardboard box.
- In case of loss, Uvener templates are available individually to purchase.

Warranty:

Dental Art Innovations Pty Ltd recognizes its responsibility to replace product proven to be defective. Dental Art Innovations Pty Ltd does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the product as described. Before using Uvener, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Life Cycle - Warning:

- ⚠ Examine Uvener templates prior to use. Do not use if a change in color, translucency or shape is noted.

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Uvener® is a registered trademark of Dental Art Innovations Pty Ltd
MedNet GmbH, Borkstrasse 10,
Ultradent Products GmbH