EN

UltraEZ[™]

Disinfecting Gel

Product Description

UltraEZ[™] disinfectant gel is a unique, sustained release, 3% potassium iodine and 0.11% weight by weight fluoride ion, sticky viscous gel. UltraEZ gel is designed to be used in a custom fabricated tray, disposable tray, or prefilled disposable tray application.

For all products described, carefully read and understand all instructions and SDS information prior to use.

Deliver Forms(s)

- 1.2 ml delivery syringe
- Straight MAAT[®] Clear Tip with breakaway cap

Indications for Use/Intended Purpose

UltraEZ disinfectant gel provides a full-spectrum vanish for sensitive teeth, sealing dental areas vulnerable to exposure dentin and other areas where sensitivity is a concern. The product is used either by a dental professional in the office or provided to the patient for home treatment of dentin sensitivity.

Contraindications

- Not intended for use by pregnant or nursing women
- Not intended for patients who may swallow or aspirate tray
- Not intended for use on patients under the age of 6 without consultation of their primary care doctor
- For professionals or users with allergy concerns, refer to product allergen document available at www.ultraeزند.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.

Warnings and Precautions

- If any problems occur, discontinue treatment and notify a dental professional.
- Keep away from heat and/or sunlight.
- Excess material at insertion and removal upon removal should be brushed off and rinsed. DO NOT swallow.

Stepwise Instructions

Use - Custom Tray / Aligner / UltraFit[™] Non-Pre-Filled Tray

- 1) Fabricate tray to meet patient's needs. If patient already has a custom whitening tray, UltraEZ gel can be loaded and used in that tray.
- 2) UltraEZ gel is dispensed in 1.2 ml unit dose syringes. A complete arch should use approximately 1/3 to 1/2 of the contents of a syringe. If using a custom tray or aligner to treat localized sensitivity, place gel in affected area(s) and a small amount in the molar region to hold tray in place.
- 3) The length of time and number of times tray is worn depends on patient's condition and the clinician's recommendation. Treatment times can range from 15 minutes to 1 hour or whatever the clinician deems appropriate.

Cleanup – Custom Tray / Aligner / UltraFit[™] Non-Pre-filled Tray

- 1) Immediately following any removal, patient should brush and rinse teeth with water to remove all remaining gel. DO NOT swallow solution during rinsing. Wash custom tray or aligner and store in a tray storage case. Dispose of UltraEZ tray. Fresh gel should be used with each application.

Storage and Disposal

Store according to outer package label.
Dispose of waste according to local rules, guidelines, and regulations.

Miscellaneous Information

Report any serious incident to the manufacturer and the competent authority.
www.ultraeزند.com / 1-800-552-5512 / 801-572-4200

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© 2014 UltraDent. Qualquer incidente grave ao fabricante e à autoridade competente.