

Sensitive?

Not anymore

UltraEZ[®] Desensitizing Gel

- Provides fast, easy treatment for sensitive smiles
- Prefilled UltraFit™ tray adapts to any smile for a comfortable fit
- Also available in syringes for use with custom trays



ULTRADENT
PRODUCTS, INC.



800.552.5512 ULTRADENT.COM



Additional features of UltraEZ desensitizing gel

- Quickly eliminates sensitivity caused by tooth whitening, abrasion, root exposure, and thermal or chemical changes
- Can be used before and after whitening treatments to dramatically reduce sensitivity



Now Featuring The **UltraFit™** Tray



Before wearing UltraFit tray in the mouth



UltraFit tray after just 10 minutes in the mouth



1008 **UltraEZ Refill**
4 x 1.2 ml UltraEZ syringes



1007 **UltraEZ Econo Refill**
20 x 1.2 ml UltraEZ syringes



5721 **UltraEZ Tray Combo (Upper/Lower)**
10 x Upper/lower trays



5743 **UltraEZ Mini Kit**
4 x Upper/lower trays



Instructions



1. Remove from package.



2. Center tray on arch.



3. Gently suck down or swallow.



4. Remove outer tray.



5. Suck down or swallow again.



6. Wear for 15 minutes to 1 hour per day or as directed by your dentist.