

Get More Confidence in Your Smile

Your smile says more about you than any other physical feature. It's the first thing people notice. Wouldn't it be great to have a simple method for whitening your smile in an affordable treatment that's convenient enough to take with you anywhere?

Good news! Now you can have the bright, white smile you've always wanted for a price that is much more affordable than you may think. Introducing Opalescence Go, the easiest, fastest, and best-tasting way to brighten your smile. Offering professional results in the convenience of pre-loaded disposable trays, Opalescence Go is perfect for patients on the go.

WARNING: Some users may experience sensitivity. If this occurs, it is most often mild and transient but in some cases may require treatment by your dentist.

If you experience significant discomfort, stop using this product and consult your dentist. If you have any questions about this product or treatment, please ask your dentist.



Rx

EN- For professional use only
DE- Nur zur Verwendung durch Sachkundige
FR- Destiné à un usage professionnel uniquement
NL- Alleen voor professioneel gebruik
IT- Esclusivamente ad uso professionale
ES- Sólo para uso profesional
PT- Apenas para uso profissional
SV- Endast för yrkesmässig användning
DA- Kun til professionel brug
FI- Vain ammattikäyttöön
EL- Για επαγγελματική χρήση μόνο
PL- Wyłącznie do zastosowań profesjonalnych



EN- Keep out of reach of children
DE- Aus der Reichweite von Kindern aufbewahren
FR- Tenir hors de la portée des enfants
NL- Buiten bereik van kinderen houden
IT- Tenere lontano dalla portata dei bambini
ES- Mantenga alejado del alcance de los niños
PT- Mantenha o produto fora do alcance de crianças
SV- Förvaras utom syn- och räckhåll för barn
DA- Opbevares utilgængeligt for børn
FI- Pidä pois lasten ulottuvilta
EL- Φυλάξτε το μακριά από τα παιδιά
PL- Przechowywać w miejscu niedostępnym dla dzieci



EN- Health hazard
DE- Gesundheitsgefahr
FR- Dangereux pour la santé
NL- Gevaar voor de gezondheid
IT- Pericolo per la salute
ES- Peligro para la salud
PT- Riscos à saúde
SV- Hälsöfara
DA- Sundhedsfarlig
FI- Terveyshäva
EL- Κινδύνους την υγεία
PL- Zagroźenie dla zdrowia



EN- Refrigerate
DE- Kühlen
FR- Réfrigérer
NL- In de koelkast bewaren
IT- Conservare in frigorifero
ES- Refrigerar
PT- Refrigerar
SV- Förvaras i kylskåp
DA- Opbevares i køleskab
FI- Säilytä jääkaapissa
EL- Διατηρήστε το στο ψυγείο
PL- Przechowywać w lodówce



EN- See instructions
DE- Siehe Anleitung
FR- Voir le mode d'emploi
NL- Zie gebruiksaanwijzing
IT- Vedere le istruzioni
ES- Lea las instrucciones
PT- Ver instruções
SV- Se bruksanvisning
DA- Se vejledningen
FI- Lue käyttöohjeet
EL- Διατείτε τις οδηγίες
PL- Patrz instrukcja



EN- Keep away from heat/sunlight
DE- Von Hitze/Sonneinstrahlung fernhalten
FR- Conserver à l'abri de la chaleur/lumière du soleil
NL- Uit de buurt van warmte/zonlicht bewaren
IT- Tenere lontano da fonti di calore/luce del sole
ES- Mantenga alejado del calor/luz solar
PT- Mantenha distância do calor/luz solar
SV- Skyddas från värme/sollys
DA- Må ikke udsættes for varme/sollys
FI- Suojattava kuumaudeilta/aurinkovoimalta
EL- Φυλάξτε το μακριά από θερμότητα/ηλιακό φως
PL- Przechowywać z dala od ciepła/promieniowania słonecznego



EN- Do not re-use to avoid cross contamination
DE- Um Kreuzkontamination zu vermeiden, nicht erneut benutzen
FR- Ne pas réutiliser pour éviter la contamination croisée
NL- Niet opnieuw gebruiken om kruisbesmetting te voorkomen
IT- Non riutilizzare per evitare la contaminazione crociata
ES- No reutilice para evitar la contaminación cruzada
PT- Não reutilize para evitar a contaminação cruzada.
SV- För undvikande av korskontamination, återanvänd inte
DA- Må ikke genbruges for at undgå krydskontaminering
FI- Älä käytä uudelleen ristikonaminaation välttämiseksi.
EL- Μην το επαναχρησιμοποιείτε για να αποφύγετε τη διασταυρούμενη μόλυνση
PL- Nie używać ponownie, aby uniknąć przeniesienia zakażenia

Manufactured by Ultradent Products, Inc.
505 West Ultradent Drive (10200 South),
South Jordan, Utah 84095
www.opalescence.com Made in USA

© Copyright 2018 Ultradent Products, Inc. All Rights Reserved.

For Professional use only.
For an SDS of this product,
please see: www.ultradent.com
ULTRADENT
PRODUCTS, INC.

68582.4 110118

INSTRUCTIONS

professional whitening to go

10% Mint Ingredients:
Glycerin, Water (aqua), Carbomer, Hydrogen Peroxide, Urea (Carbamide) Peroxide, Sodium Hydroxide, PVP, Silica, Xylitol, Disodium Phosphate, Potassium Nitrate, Sucralose, Flavor (aroma), Sodium Fluoride, EDTA, Sodium Lauryl Sulfate

10% Melon Ingredients:
Glycerin, Water (aqua), Carbomer, Hydrogen Peroxide, Urea (Carbamide) Peroxide, Sodium Hydroxide, PVP, Silica, Flavor (aroma), Xylitol, Disodium Phosphate, Potassium Nitrate, Sucralose, Sodium Fluoride, EDTA, Sodium Lauryl Sulfate

15% Mint Ingredients:
Water (Aqua), Glycerin, Hydrogen Peroxide, Carbomer, Urea (Carbamide) Peroxide, Sodium Hydroxide, PVP, Potassium Nitrate, Silica, Xylitol, Disodium Phosphate, Sucralose, Flavor (Aroma), Sodium Fluoride, EDTA, Sodium Lauryl Sulfate



INSTRUCTIONS



1. Remove from package
'U' • Upper bleaching tray 'L' • Lower bleaching tray.



2. Center tray on arch.



3. Gently suck down or swallow.



4. Remove outer tray.



5. Suck down or swallow again.



6. Wear 10% for 30 to 60 minutes and 15% for 15 to 20 minutes.

PRECAUTIONS

1. Keep tooth bleaching product away from small children.
2. A dental examination is recommended prior to bleaching. Some discolorations may indicate the need for dental treatment other than bleaching. Porcelain crowns, veneers, and white fillings do not lighten upon bleaching.
3. Only intended for use by those with healthy gum tissue. Gum tissue irritation may result if used in individuals with periodontal disease.
4. This product is NOT for nighttime use.
5. Occasionally, some parts of teeth may bleach faster or less evenly than others. Results may be varied. In approximately 5% of the population, teeth may be resistant to bleaching. Bleaching teeth with large metal fillings may cause the metal color to show through the teeth.
6. Acidic foods/juices may cause sensitivity if consumed shortly after bleaching.
7. Do not use tobacco products or eat while bleaching.
8. Not intended for use by pregnant women. Consult a health care practitioner if you are breast feeding.
9. Keep gel out of heat/sunlight. Do not freeze.
10. Do not swallow gel or rinses. Avoid direct contact of the active surface of the tooth whitening product with the gums and/or salivary flow. Product contains peroxide and may contain fluoride; swallowing large amounts can be harmful.
11. Avoid contact with eyes.