

Dentist-designed at-Home whitening



Our cutting-edge technology lines the tray with whitening gel for precise placement, so patients just need to position the trays on their teeth and wear for the recommended time.

Opalescence™
Home
Advanced

Your patients want a simple way to get a brighter smile they are confident in. Now, you can offer them professional, mess-free whitening in one easy step with our new Opalescence Home Advanced prefilled trays.



Brighter Smiles. Better Days.



- Professional results in a one-step treatment
- Prefilled Ultrafit™ tray is ready to use right out of the box, conforming to any patient's smile
- Whitening gel is distributed evenly on the tray, delivering it exactly where needed
- Whitens teeth up to 4–5 shades for quick, visible results¹

Procedure*



Scan to see the how-to video



1. Remove product from packaging.
'U' – Upper whitening tray
'L' – Lower whitening tray



2. Position upper tray on teeth.



3. Bite firmly, then suck on tray for 2 seconds. Do not press tray against teeth as it pushes the whitening gel against the gums, causing tissue irritation.



4. Repeat the process for the lower tray.



5. After indicated wear time, remove whitening trays and brush teeth.



10%: 30–60 min.



15%: 15–20 min.

*See IFU for complete instructions and precautions



Opalescence Home Advanced Patient Kits Case of 6

Flavor	10%	15%
Mint	5821-US	5822-US

10 x Each upper/lower trays in each kit



Opalescence Home Advanced Mini Kits Case of 12

Flavor	10%	15%
Mint	6005-US	6006-US

4 x Each upper/lower trays in each kit



Opalescence Home Advanced Sample Tower

Flavor	10%	15%
Mint	6003-US	6004-US

20 x Each upper/lower trays with instructions in each sample



SCAN TO VIEW PRICING

¹. Cordeiro D, Toda C, Hanan S, et al. Clinical evaluation of different delivery methods of at-home bleaching gels composed of 10% hydrogen peroxide. *Oper Dent.* 2019;44(1):13–23. doi:10.2341/17-174-C.