## EN UltraCal™ XS Calcium Hydroxide Paste

**Description:** UltraCal™ XS is an aqueous, 12.5 pH, syringeable, radiopaque, calcium hydroxide paste. Use it as an endodontic treatment dressing. Superior delivery control of UltraCal™ is obtained with a Navitip™ attached.

**Indications:** UltraCal™ XS to be used as per literature teachings for pulp capping, lining pulpotoomies, or as a temporary dressing as per apification procedures in endodontics.

**Procedure:** Following instrumentation, remove luer lock cap from UltraCal™ XS syringe and attach a Navitip. Deliver UltraCal™ XS short of the apex while slowly withdrawing tip.

- Precautions:**
- 1) Material is basic. Use carefully and rinse with lots of water if material is inadvertently discharged in an undesired area.
  - 2) Tips are disposable.
  - 3) Verify the color and consistency of the product flow before applying by expressing a small amount onto a mixing pad or gauze. Discard any clear liquid that is expressed.
  - 4) Deliver with Navitip, staying at least 2mm shy of the apex, always

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