EN	Cleaning & Sterilization Processing Instructions							
Initial treatment at the time of use	Sterilization is required before every use.							
Preparation before cleaning	Wipe device with 70% IPA soaked gauze.							
Cleaning	Place device in an ultrasonic cleaner with the detergent listed below:							
	Detergent	Cleaning Time	Henry Schein General Purpose Cleaner or equivalent product	2-10 mins				
Detergent	Cleaning Time							
Henry Schein General Purpose Cleaner or equivalent product	2-10 mins							
Rinsing	Rinse device with warm water to remove detergent for 1-2 mins.							
Drying	Dry with gauze. <p>Air dry 30 mins.</p>							
Maintenance, inspection and testing	Visually inspect that the device is not bent or damaged and free of debris. If debris is present repeat cleaning process.							
Packaging	Place selected device in approved sterilization packaging (ISO11407 & ISO11607).							
Sterilization	Place sterilization package in autoclave using the following settings:							
	Cycle Type	Minimum Sterilization Exposure Time	Minimum Sterilization Exposure Temperature	Minimum Dry Time	Pre-vacuum	3 mins	132°C /270°F	20 mins
	Cycle Type	Minimum Sterilization Exposure Time	Minimum Sterilization Exposure Temperature	Minimum Dry Time				
Pre-vacuum	3 mins	132°C /270°F	20 mins					
Storage	Keep device in sterilized packaging until ready for use. Keep out of sunlight and moisture.							
Additional information	If sterilization packaging is compromised, repeat sterilization process. This procedure was validated by an independent and accredited laboratory.							

To obtain a printed copy of these instructions call +1.801.553.4057

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