



ADDENDUM TO CLEANING PROCEDURE FOR V4 RINGS:

1. Post Class II procedure, prompt cleaning of rings is essential.
2. To free hard-to-remove composite residue that may be adhered to tines, gently lift off the material using a soft, blunt plastic instrument such as a composite spatula.
3. Wash rings thoroughly with warm water and a soft-bristled brush to remove ANY chemical residue. Alternatively, a soft cloth soaked in warm water could be used to wipe the tines.
4. If using an ultrasonic cleaner, place rings in a beaker of water (**isolated from any other instruments and any ultrasonic cleaning solutions/detergents**).
5. Place in steam autoclave at a maximum temperature of 134 deg C for a minimum of 3 minutes.

