Amelogen® Plus Singles Unit dose tips (0.3 grams) Composite Restorative Material

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IT

Nota: Amelogen Plus composite restorative material Singles fit most composite guns.

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Directions for Use:

1. Si refrigerado, bring product to room temperature.

2. Apply bonding agent to preparation according to manufacturer's instructions.

3. OPTIMELLE: Place a thin layer of flowable composite.

4. Remove cap from Amelogen Plus composite syringe. If desired, use the Quad Key to remove inside flanges for open barrel delivery.

5. Twist syringe to express composite. Restorative will lessen with each twist.

6. To apply incrementally, remove incremental pieces of composite, express the composite slightly above the inner white KleinSleevé.

7. Place Amelogen Plus composite restorative material in incremental layers applying the dentin primer approximately 0.5 - 1.5mm thick and cure. Follow with 2mm increments curing each layer with VALO or another high quality curing light ($600mW/cm^2$), cure each layer 20 seconds. For lights with an output of $400mW/cm^2$, cure for 40 seconds.

NOTA: To avoid cross-contamination, use a clean instrument with each increment removal.

8. After final increment is placed, sculpt and light cure.

9. Shape and define the final anatomy using finishing diamonds or multi-fluted carbide burs. Finish and polish with polishing discs, cups, points or burs. A composite restorative can be polished to a mirror finish according to manufacturer's instructions to increase surface hardness and reduce surface wear.

Precautions and Warnings:

1. Read and understand all instructions before using product.

2. For use at room temperature. Refrigerate bulk storage.

3. Amelogen Plus composite capsule are single use only. Do not reuse to avoid the risk of cross contamination.

4. Use a quality curing light to ensure maximum physical properties of the composite. Uncured material will result in inferior restoration and may also cause pulpal irritation.

5. Clinician and patient should wear UV protective eyewear when curing composite.

6. To prevent the pre-polymerization of the composite, minimize exposure to bright lights during the curing process.

7. To avoid bond to each succeeding layer, do not remove oxygen inhibition layer between incremental layers of composite.

8. Ensure composite is totally polymerized between layers and interproximally.

9. Not intended for intradental use.

10. Avoid cross-contamination. Avoid repeated contact of uncured dental resin with skin. Do not use on patients with known sensitivity to acrylates or other resins. If allergic reaction, dermatitis or rash develops, consult a physician.

11. Proper disposal of capsule after use.

12. Keep away from direct sunlight.

13. Do not use after expiration date noted on containers.

14. Keep out of reach of children.

Note: For SDS and additional information about using Amelogen Plus composite and other related products, please go to www.ultradent.com

Amelogen® Plus Singles Puntas de una dosis (0.3 gramos) Composite Restorative Material

Amelogen® Plus Singles Pontas doseadoras (0.3 grama) Composite Restorative Material

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