

# Ultra-Blend™ *plus*

Cavity Liner and Pulp-Capping Material




Ultra-Blend plus liner is a unique dentin-adhering liner and base material containing calcium hydroxide in a urethane dimethacrylate (UDMA) base. It's perfect for pulp capping and will not dissolve over time. Made of a light-activated, radiopaque material, Ultra-Blend plus liner is highly filled for minimal shrinkage.

- Bioactive<sup>1</sup> liner and pulp-capping material
- Superior calcium release<sup>2</sup>
- Controlled, precise syringe delivery
- No mixing necessary
- Light-cured
- Radiopaque

# Ultra-Blend™ plus


## Ultra-Blend™ plus liner application:

A bioactive base material is an essential tool used in restorative dentistry—especially in cases where the pulp is exposed or near exposure. Thanks to its superior calcium release, Ultra-Blend plus liner effectively stimulates the reparative process.<sup>1</sup> That's one reason it's been a preferred pulp-capping material for over 20 years.

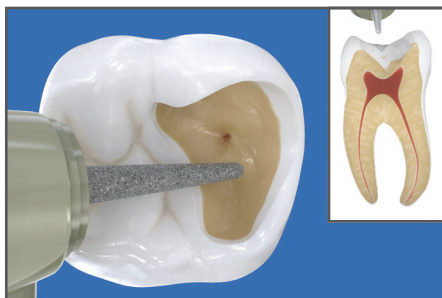


### Small Exposure

Use Ultra-Blend plus liner near pulp (pink) and for small nonhyperemic exposure. For larger exposure and/or hyperemic pulp, endodontic therapy should be considered.

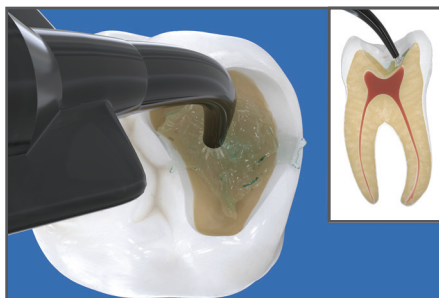

### Consepsis™ Solution

Apply Consepsis™ antibacterial solution with plastic Blue Mini™ Dento-Infusor™ or Black Mini™ brush tip for 60 seconds. Let sit. Do not scrub. Air dry.

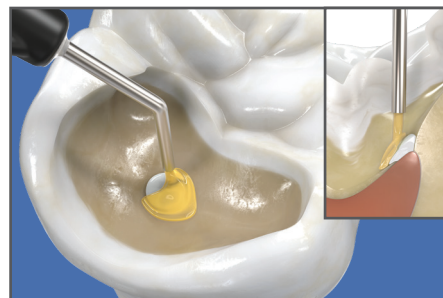


### MTAFlow™ Cement

Optional: If the exposure is larger than a pinhole, apply MTAFlow™ cement onto and slightly around the pulp exposure. Remove excess material with a dry cotton pellet. When pulpal floor is clean and dry, place Ultra-Blend plus liner, overlapping slightly onto sound dentin to maximize surface area for dentin bonding. If the pulp is hyperemic, endodontic therapy should be considered.



### Ultra-Blend plus Liner

With Black Micro™ tip, apply Ultra-Blend plus liner to dry dentin for direct or indirect pulp caps and light cure. Minimize dentin coverage to maximize available dentin for bonding.

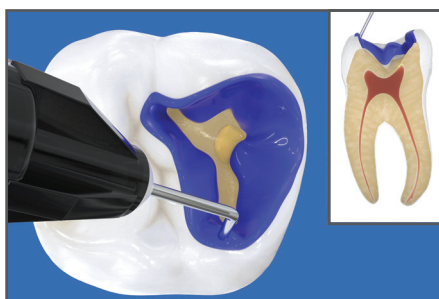

### Ultra-Etch™ Etchant

Apply Ultra-Etch™ 35% phosphoric acid etchant solution for 15 seconds. Suction, rinse, and dry until damp. **NOTE:** If desired, apply Consepsis solution prior to bonding, then place again for 60 seconds. Dry until dentin is slightly moist and proceed to the bonding agent.



### Dentin Bonding/Peak™ Universal Bond Adhesive

With the Inspirall™ Brush tip, apply Peak™ Universal Bond adhesive, paint onto enamel and scrub into dentin for 10 seconds. Air thin at half pressure for 10 seconds and light cure for 10 seconds with the VALO™ curing light on Standard Power mode. Restore with a quality composite.



#### 415 - Ultra-Blend plus Syringe Kit

2 x 1.2 ml Dentin syringes  
2 x 1.2 ml Opaque White syringes  
20 x Black Micro tips  
20 x Black Mini tips



416 - Ultra-Blend plus Dentin Syringe 4pk  
417 - Ultra-Blend plus Opaque White Syringe 4pk  
1.2 ml syringes



SCAN  
TO VIEW  
PRICING



© 2023 Ultradent Products, Inc. All rights reserved.  
15706AR12 093022

800.552.5512 | ULTRADENT.COM

