


# Ultra-Blend™ plus

Cavity Liner and Pulp-Capping Material

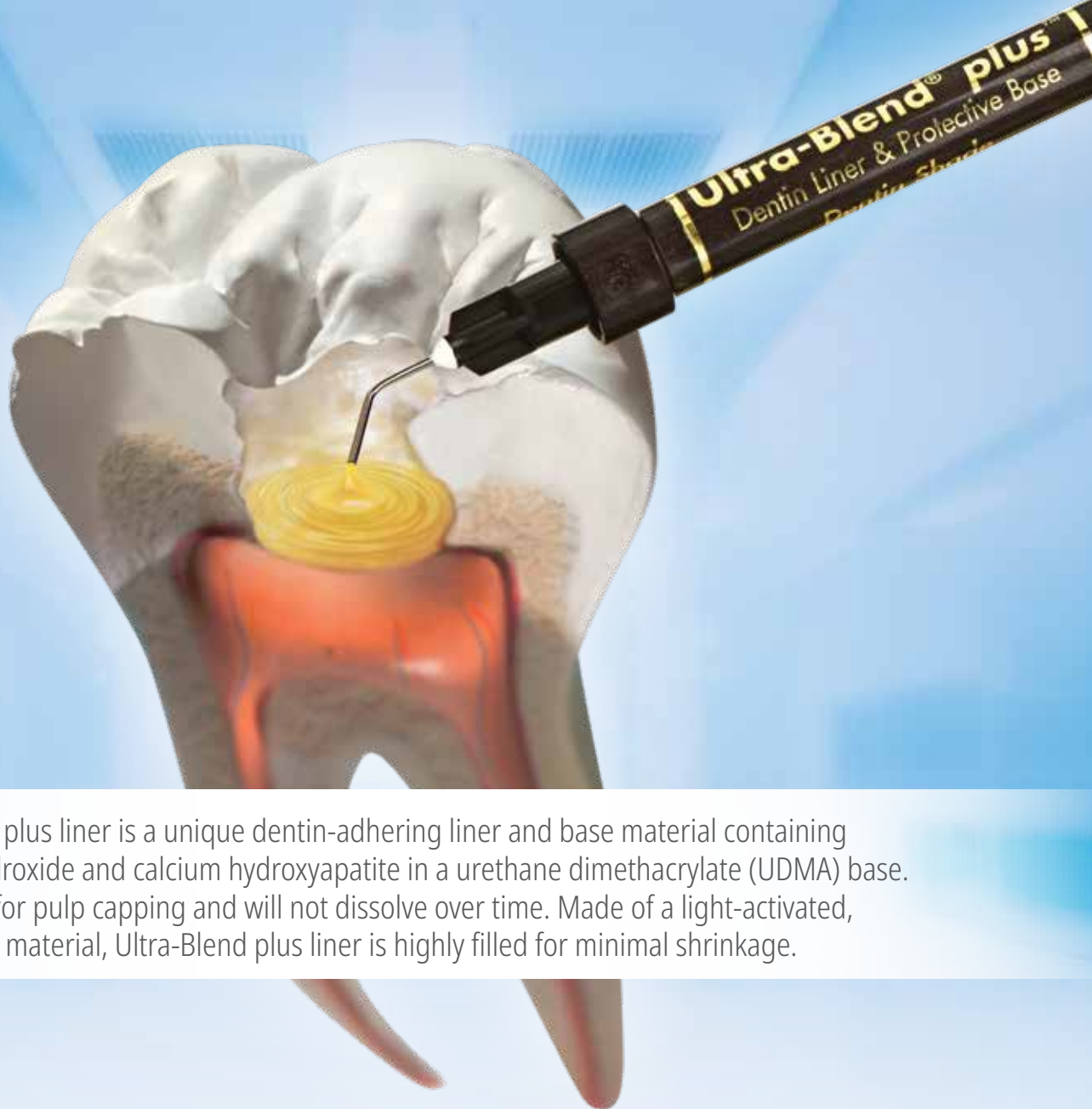

Ultra-Blend plus liner is a unique dentin-adhering liner and base material containing calcium hydroxide and calcium hydroxyapatite in a urethane dimethacrylate (UDMA) base. It's perfect for pulp capping and will not dissolve over time. Made of a light-activated, radiopaque material, Ultra-Blend plus liner is highly filled for minimal shrinkage.

- Bioactive<sup>1</sup> liner and pulp-capping material
- Superior calcium release<sup>2</sup>
- Controlled, precise syringe delivery
- No mixing necessary
- Light cured when clinician is ready
- Radiopaque

# Ultra-Blend™ plus

## Ultra-Blend™ plus liner application:


A bioactive base material is an essential tool used in restorative dentistry—especially in cases where the pulp is exposed. Thanks to its superior calcium release, Ultra-Blend plus liner effectively stimulates the reparative process.<sup>1</sup> That's one reason it's been a preferred pulp-capping material for over 15 years.




Use Ultra-Blend plus liner near pulp (pink) and for small nonhyperemic exposure. For larger exposure and/or hyperemic pulp, initiate endodontic therapy.




Apply Consepsis™ antibacterial solution with plastic Blue Mini™ Dento-Infusor™ or Black Mini™ brush tip for 60 seconds with gentle scrubbing action. Dry thoroughly. Do not rinse.

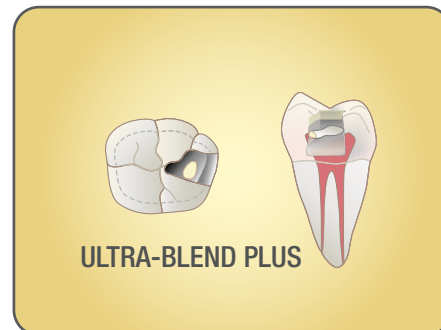

With Black Micro™ tip, apply Ultra-Blend plus liner to dry dentin for direct or indirect pulp caps and light cure. Minimize dentin coverage to maximize available dentin for bonding.

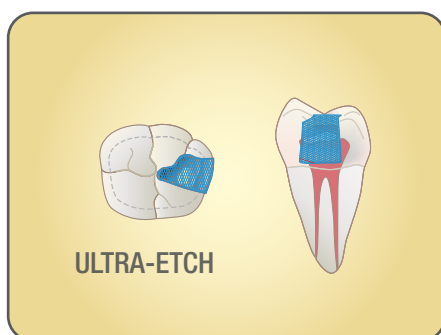

Apply Ultra-Etch™ 35% phosphoric acid etchant solution for 15 seconds. Suction, rinse, and dry until damp.

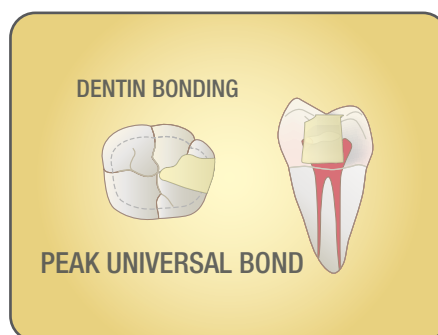

With the Inspiral™ Brush tip, apply Peak™ Universal Bond adhesive, paint onto enamel and scrub into dentin for 10 seconds. Air thin at half pressure for 2–3 seconds and light cure for 10 seconds with the VALO™ curing light on Standard Power mode. Restore with a quality composite.



### 415 - Ultra-Blend plus Syringe Kit

2 x 1.2 ml Dentin syringes  
2 x 1.2 ml Opaque White syringes  
20 x Black Micro tips  
20 x Black Mini tips



416 - Ultra-Blend plus Dentin Syringe 4pk  
417 - Ultra-Blend plus Opaque White Syringe 4pk  
1.2 ml syringes

