

Ultra-Blend™ plus

Cavity Liner and Pulp-Capping Material




Ultra-Blend plus liner is a unique dentin-adhering liner and base material containing calcium hydroxide in a urethane dimethacrylate (UDMA) base. It's perfect for pulp capping and will not dissolve over time. Made of a light-activated, radiopaque material, Ultra-Blend plus liner is highly filled for minimal shrinkage.

- Bioactive¹ liner and pulp-capping material
- Superior calcium release²
- Controlled, precise syringe delivery
- No mixing necessary
- Light-cured
- Radiopaque

Ultra-Blend™ plus


Ultra-Blend™ plus liner application:

A bioactive base material is an essential tool used in restorative dentistry—especially in cases where the pulp is exposed or near exposure. Thanks to its superior calcium release, Ultra-Blend plus liner effectively stimulates the reparative process.¹ That's one reason it's been a preferred pulp-capping material for over 20 years.

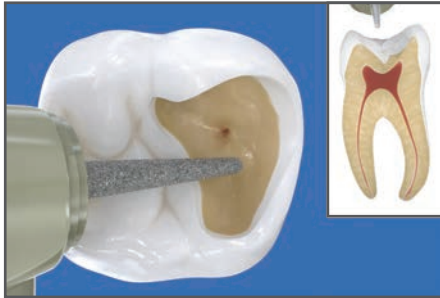


Small Exposure

Use Ultra-Blend plus liner near pulp (pink) and for small nonhyperemic exposure. For larger exposure and/or hyperemic pulp, endodontic therapy should be considered.

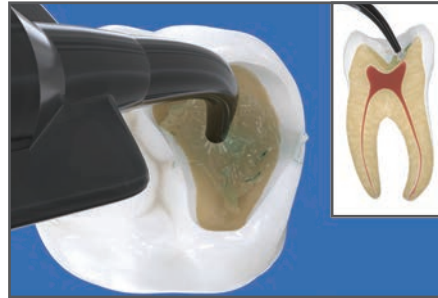

Consepis™ Solution

Apply Consepis™ antibacterial solution with plastic Blue Mini™ Dento-Infusor™ or Black Mini™ brush tip for 60 seconds. Let sit. Do not scrub. Air dry.

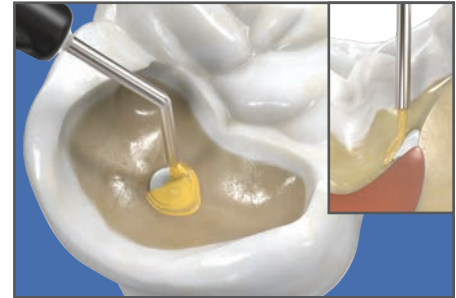


MTAFlow™ Cement

Optional: If the exposure is larger than a pinhole, apply MTAFlow™ cement onto and slightly around the pulp exposure. Remove excess material with a dry cotton pellet. When pulpal floor is clean and dry, place Ultra-Blend plus liner, overlapping slightly onto sound dentin to maximize surface area for dentin bonding. If the pulp is hyperemic, endodontic therapy should be considered.



Ultra-Blend plus Liner

With Black Micro™ tip, apply Ultra-Blend plus liner to dry dentin for direct or indirect pulp caps and light cure. Minimize dentin coverage to maximize available dentin for bonding.

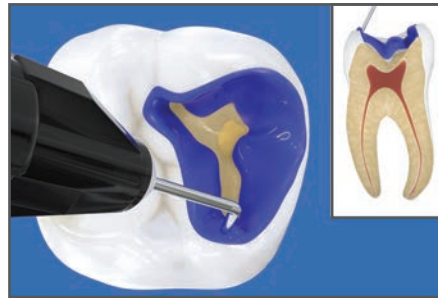

Ultra-Etch™ Etchant

Apply Ultra-Etch™ 35% phosphoric acid etchant solution for 15 seconds. Suction, rinse, and dry until damp. *NOTE: If desired, apply Consepis solution prior to bonding, then place again for 60 seconds. Dry until dentin is slightly moist and proceed to the bonding agent.*



Dentin Bonding/Peak™ Universal Bond Adhesive

With the Inspiral™ Brush tip, apply Peak™ Universal Bond adhesive, paint onto enamel and scrub into dentin for 10 seconds. Air thin at half pressure for 10 seconds and light cure for 10 seconds with the VALO™ curing light on Standard Power mode. Restore with a quality composite.



415 - Ultra-Blend plus Syringe Kit

2 x 1.2 ml Dentin syringes
2 x 1.2 ml Opaque White syringes
20 x Black Micro tips
20 x Black Mini tips



416 - Ultra-Blend plus Dentin Syringe 4pk
417 - Ultra-Blend plus Opaque White Syringe 4pk
1.2 ml syringes



SCAN
TO VIEW
PRICING