Opal™ Band EN
Orthodontic Band Cement

Description:
Opal™ Band cement is a light blue, two-part, triple cure, resin-based glass ionomer orthodontic cement.

Indications:
This product uses chemical, light, and glass ionomer polymerization to cement all types of orthodontic bands to the teeth.

Instructions for Use:
1. After determining proper band fit, prophyl, and rinse the tooth leaving the enamel slightly damp.
Optional: lightly roughen the inside of the band with a fine diamond bur or microetcher.
2. Remove cap from dual barrel syringe by twisting counter-clockwise and discard.
NOTE: Prior to attaching tip, verify uniform flow from both syringe orifices by expressing a small amount onto a mixing pad. Use caution not to cross-contaminate orifices. The used mixing tip remains in place as the cap until a new mixing tip is attached prior to reuse. Discard old mixing tip when removed.
3. Attach mixing tip by lining up internal cap stems with syringe orifices. Twist clockwise to lock and verify flow.
4. Place desired amount to the interior surface of the band.
5. Seat band onto the tooth and clean excess cement with a cotton roll or gauze.
6. Light cure the occlusal surface with VALO using 2 cycles of a 3-second plasma or 20 seconds for lights with output <600mW/cm².
7. Band cement will chemically cure after 10 minutes if light is not used.
NOTE: In higher force situations (RPE or similar fixed appliances), etching the enamel or conditioning with a self-etching primer prior to band placement will significantly increase the adhesion.

Precautions and warnings:
1. Carefully read and understand all instructions before use.
2. Review all precautions on individual product inserts before use.
3. Use only recommended tips.
4. Refrigerate band cement when not in use.
5. If allergic reaction, dermatitis, or rash develop, consult a physician.
6. Dispose of tips and empty syringes properly.
7. When using this product, do not remove mixing tip until next use.
8. Disinfect syringe between uses.
Review instructions, warnings, precautions, and SDS before beginning treatment. For SDS and additional information about our products, please go to www.ultradent.com.

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