PermaFlo™ Pink <p>Flowable Composite</p>	EN
<div><b>Product Description</b></div> <p>PermaFlo™ Pink composite is a light-cured, radiopaque, methacrylate based flowable composite. PermaFlo Pink composite contains 1µm average particle size with narrow upper limit particle distribution. It is thixotropic, 68% filled. For all products described, carefully read and understand all instructions and SDS information prior to use.</p> <div><b>Delivery Form(s)</b></div> <ul style="list-style-type: none"><li>1.2 ml delivery syringe</li> <li>Delivery tip: <ul style="list-style-type: none"><li>Micro 20 ga tip</li></ul></li></ul>	
<div><b>Indications for Use/Intended Purpose</b></div> <p>Restorative composites are used for direct and indirect restorations in both the anterior and posterior regions.</p> <div><b>Contraindications</b></div> <ul style="list-style-type: none"><li>Do not use on patients with known sensitivity to methacrylates or other resins. If dermatitis, rash, or other allergic reaction is observed, rinse exposed area thoroughly and have patient consult their physician.</li> <li>Do not use on patients with allergic concerns, refer to product allergen information available at <a href="http://www.ultradent.com">www.ultradent.com</a>. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.</li></ul> <div><b>Warnings and Precautions</b></div> <ul style="list-style-type: none"><li>Resins can be sensitizing. Do not use on patients with a known sensitivity to acrylates or other resins. If allergic reaction, dermatitis, or rash develops, have patient consult their physician.</li> <li>To avoid cross-contamination, use clean gloves and a clean instrument.</li> <li>Redirect overhead light to prevent premature polymerization.</li> <li>To optimize bond to each succeeding layer, do not remove oxygen inhibition layer between incremental layers of composite.</li> <li>Ensure composite is totally polymerized between layers; partially cured material could cause pulpal irritation and will have deficient physical properties.</li></ul>	
<div><b>Stepwise Instructions</b></div> <div><b>Preparation</b></div> <ol style="list-style-type: none"><li>Remove luer lock cap and securely attach a recommended delivery tip onto syringe.</li> <li>Verify flow and consistency prior to applying intraorally. If resistance is met, replace tip and try again.</li> <li>Isolate preparation with retraction cord and/ or rubber dam to control sulcular fluid.</li> <li>Etc) according to manufacturer's instructions.</li> <li>Apply bonding agent to preparation according to manufacturer's instructions.</li></ol> <div><b>Use</b></div> <ol style="list-style-type: none"><li>Apply first layer of PermaFlo Pink composite (0.5–1.5 mm) and light cure. Follow with 2 mm increments, curing after each increment. For curing lights with an intensity of ≥800 mW/cm2 in the wavelength region of 385–515 nm, cure each layer for 10 seconds. A final cure of 20 seconds is recommended.</li> <li>Shape and finish using preferred method.</li></ol> <div><b>Processing</b></div> <p>To avoid cross-contamination, wipe syringe with an intermediate-level disinfectant between uses and use a disposable barrier sleeve. If a barrier sleeve is not used, syringe should be treated as single-use.</p> <div><b>Storage and Disposal</b></div> <p>Dispose of waste according to local rules, guidelines, and regulations.</p> <div><b>Miscellaneous Information</b></div> <p>Report any serious incident to the manufacturer and the competent authority. <p><a href="http://www.ultradent.com/">www.ultradent.com/</a> +1-800-552-5512 / 801-572-4200</p></p>	

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