■ **ENI Description and Indications for Use:** FORMA™ composite contains bis-GMA, TEGDMA, Bis-EMA and UDMA. FORMA restorative composite is designed for restorations in anterior and posterior teeth. Its formula contains a zirconia/silica inorganic filler combination and barium glass. All shades of FORMA composite are radiopaque and based on Vita® shade guide.

List of supporting materials accompanying the product:

The respective Instructions for Use manual accompanies the product.

FORMA Syringe

Directions for use:

1. Apply dentin bonding agent to preparation according to manufacturer's instructions.
2. **OPTIONAL:** Place an adaptive layer of flowable composite before placing FORMA composite.
3. Flip open cap on the FORMA composite syringe.
4. Twist handle clockwise (stem) to express composite. Resistance will lessen with each twist.
5. Place FORMA in incremental layers applying the first layer approximately 0.5 – 1.5 mm thick and cure. Follow with up to 2 mm increments curing after each increment. If curing with VALO curing light or another high quality curing light (>800mW/cm²), cure each layer 20 seconds. For lights with an output <800mW/cm², cure for 40 seconds.
- NOTE:** to avoid cross-contamination, use a clean instrument every time you remove the composite from the syringe.
6. After final increment is placed, sculpt and light cure.
7. Shape and define the final anatomy using finishing diamonds or multi-fluted carbide burs. Finish and polish with polishing disks, cups, points or brushes.

Precautions and warnings:

1. Read and understand all instructions before using product.
2. Store at temperature within 2–27°C.
3. Use a quality curing light to ensure maximum physical properties of the composite.

- Uncured material will result in an inferior restoration and may also cause pulpal irritation.
4. Clinician and patient should wear UV and blue light protective eyewear when curing composite.
 5. To lessen pre-polymerization of the composite, minimize exposure to bright lights during the procedure.
 6. To optimize bond to each succeeding layer, do not remove oxygen inhibition layer between incremental layers of composite.
 7. Ensure composite is totally polymerized between layers and interproximally.
 8. Syringes are not for intraoral use.
 9. Resins can be sensitizing. Avoid repeated contact of uncured dental resin with skin. Do not use on patients with a known sensitivity to acrylates or other resins. If allergic reaction, dermatitis or rash develops, consult a physician.
 10. To avoid pre-polymerization of the composite by ambient light, make sure to always cover the tip of the syringe with its cap after composite has been removed.
 11. To avoid cross-contamination, use a clean instrument every time you remove the composite from the syringe.
 12. Disinfect contaminated syringes with an intermediate level disinfectant between uses.
 13. Properly dispose of syringes, according to current legislation.
 14. Keep away from extreme heat and direct sunlight.
 15. Do not use after expiration date noted on containers.
 16. Keep out of reach of children.

NOTE: for product SDS and additional information about FORMA composite, please see website: www.ultradent.com.br.

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