Cleaning & Sterilization Processing Instructions				EN
Initial treatment at the time of use		Sterilization is required before every use.		
Preparation before cleaning		Wipe device with 70% IPA soaked gauze.		
Cleaning	Place device in an ultrasonic cleaner with the detergent listed below:			
	Detergent	Cleaning Time		
	Henry Schein General Purpose Cleaner or equivalent product		2-10 mins	
Rinsing		Rinse device with warm water to remove detergent for 1-2 mins.		
Drying		Dry with gauze. Air dry 30 mins.		
Maintenance, inspection and testing		Visually inspect that the device is not bent or damaged and free of debris. If debris is present repeat cleaning process.		
Packaging		Place selected device in approved sterilization packaging (ISO11407 & ISO11607).		
		Place sterilization package in autoclave using the following settings:		
Sterilization	Cycle Type	Minimum Sterilization Exposure Time	Minimum Sterilization Exposure Temperature	Minimum Dry Time
	Pre-vacuum	3 mins	132°C /270°F	20 mins
		Do not overload the sterilizer.		
Storage		Keep device in sterilized packaging until ready for use. Keep out of sunlight and moisture.		
Additional information		If sterilization packaging is compromised, repeat sterilization process. This procedure was validated by an independent and accredited laboratory.		

To obtain a printed copy of these instructions call +1.801.553.4057

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