

Opalescence



Tooth Whitening



Powerful,  
professional whitening  
to go!





Opalescence Go whitening features ready-to-go convenience with the UltraFit™ tray for an even more adaptable and comfortable whitening experience.

- Unique tray material offers a remarkably comfortable fit and easily conforms to any patient's smile
- Molar-to-molar coverage ensures the gel comes in contact with more posterior teeth
- UltraFit tray's superior adaptation ensures the maximum amount of gel stays in contact with the teeth during whitening
- Available in 10% hydrogen peroxide (with wear times from 30–60 minutes) and 15% hydrogen peroxide (with wear times from 15–20 minutes)

### Opalescence Go Patient Kits Case of 6

| Flavor | 10%     | 15%     |
|--------|---------|---------|
| Mint   | 4635-US | 4638-US |
| Melon  | 4636-US | —       |

10 x Each upper/lower trays in each kit



### Opalescence Go Mini Kits Case of 12

| Flavor | 10%     | 15%     |
|--------|---------|---------|
| Mint   | 4645-US | 4648-US |
| Melon  | 4646-US | —       |

4 x Each upper/lower trays in each kit



### Opalescence Go Sample Dispenser Kits

| Flavor | 10%     | 15%     |
|--------|---------|---------|
| Mint   | 4893-US | 4894-US |
| Melon  | 4890-US | —       |

20 x Each upper/lower trays in each kit  
20 x Patient instructions



### Instructions



1. Remove product from packaging.  
"U" - Upper whitening tray  
"L" - Lower whitening tray



2. Position upper tray on teeth.



3. Bite firmly, then suck on tray for 2 seconds.



4. Remove colored outer tray, leaving the white inner tray on teeth. Repeat the process for the lower tray.



5. After indicated wear time, remove whitening trays and brush teeth.

Refrigeration is recommended for all Opalescence Go whitening products.

