



# Give your patients more reasons to smile

Opalescence  
● ● ● go™  
Tooth Whitening

A brighter smile can give your patients the confidence to smile more. And with Opalescence Go whitening, patients can whiten their teeth when they want, where they want. That's something to smile about.



# Powerful, Professional Whitening to Go

- Unique tray material offers a remarkably comfortable fit and easily conforms to any patient's smile
- Molar-to-molar coverage ensures the gel comes in contact with more posterior teeth
- UltraFit tray's superior adaptation ensures the maximum amount of gel stays in contact with the teeth during whitening
- Available in 10% hydrogen peroxide (with wear times from 30–60 minutes) and 15% hydrogen peroxide (with wear times from 15–20 minutes)
- Whitens up to 4–5 shades for quick, visible results<sup>1</sup>



## Opalescence Go Patient Kits Case of 6

| Flavor | 10%     | 15%     |
|--------|---------|---------|
| Mint   | 4635-US | 4638-US |
| Melon  | 4636-US | —       |

10 x Each upper/lower trays in each kit



## Opalescence Go Mini Kits Case of 12

| Flavor | 10%     | 15%     |
|--------|---------|---------|
| Mint   | 4645-US | 4648-US |
| Melon  | 4646-US | —       |

4 x Each upper/lower trays in each kit



## Opalescence Go Sample Dispenser Kits

| Flavor | 10%     | 15%     |
|--------|---------|---------|
| Mint   | 4893-US | 4894-US |
| Melon  | 4890-US | —       |

20 x Each upper/lower trays in each kit  
20 x Patient instructions

Refrigeration is recommended for all Opalescence Go whitening products.

## Procedure

Carefully read instructions and warnings delivered with products before using them.



Remove product from packaging.  
"U" - Upper whitening tray  
"L" - Lower whitening tray



Position upper tray on teeth.



Bite firmly, then suck on tray for 2 seconds.



Remove colored outer tray, leaving the white inner tray on teeth. Repeat the process for the lower tray.



After indicated wear time, remove whitening trays and brush teeth.



SCAN  
TO VIEW  
PRICING

