

Opalescence

● ● ● go™  
Tooth Whitening

ULTRADENT

Powerful,  
professional whitening  
to go!



before

after



Courtesy of Carol Jent, RDH.



Opalescence Go teeth whitening features ready-to-go convenience with the UltraFit™ tray for an even more adaptable and comfortable whitening experience.

- Unique tray material offers a remarkably comfortable fit and easily conforms to any patient's smile
- Molar-to-molar coverage ensures the gel comes in contact with more posterior teeth
- UltraFit tray's superior adaptation ensures the maximum amount of gel stays in contact with the teeth during whitening
- Available in 10% hydrogen peroxide (with wear times from 30–60 minutes) and 15% hydrogen peroxide (with wear times from 15–20 minutes)
- Whitens up to 4–5 shades for quick, visible results<sup>1</sup>

### Opalescence Go Patient Kits Case of 6

Flavor	10%	15%
Mint	4635-US	4638-US
Melon	4636-US	—

10 x Each upper/lower trays in each kit



### Opalescence Go Mini Kits Case of 12

Flavor	10%	15%
Mint	4645-US	4648-US
Melon	4646-US	—

4 x Each upper/lower trays in each kit



### Opalescence Go Sample Dispenser Kits

Flavor	10%	15%
Mint	4893-US	4894-US
Melon	4890-US	—

20 x Each upper/lower trays in each kit  
20 x Patient instructions



Refrigeration is recommended for all Opalescence Go whitening products.

### Procedure



1. Remove product from packaging.  
"U" - Upper whitening tray  
"L" - Lower whitening tray



2. Position upper tray on teeth.



3. Bite firmly, then suck on tray for 2 seconds.



4. Remove colored outer tray, leaving the white inner tray on teeth. Repeat the process for the lower tray.



5. After indicated wear time, remove whitening trays and brush teeth.

Carefully read instructions and warnings delivered with products before using them.



SCAN  
TO VIEW  
PRICING