Opalustre™ Enamel Micro-Abrasion/Slurry EN

Product Description
Opalustre is a chemical and mechanical abrasion slurry containing 6.6% hydrochloric acid and silicon carbide micro particles that can be rinsed off with water. For all instructions, carefully read and understand all described and SDS information prior to use.

Delivery Forms)

- 1.2 ml syringe
- Delivery tip
- White Mac™ Tip
- OpalCups™
- Bristle
- Ersterhandling

Use
1) Apply Opalustre layer over the area to be treated.
2) Using an OpalCups Bristle at a slow RPM (approximately 500 RPM), apply intermittent medium to heavy pressure for approximately 60 seconds per application.
3) Remove the paste with suction then rinse, evaluate and repeat every 0.2mm has been removed, and repeat as necessary.

Cleanup
1) After final rinse, remove rubber dam, or OpalDam/ OpalDam Green thoroughly.
2) Use the OpalCups Finishing for surface polishing. A fluoride treatment is also recommended.

Contraindications
• Do not use on severe fluorosis.
• Do not use on thin enamel.
• For patients or users with allergy concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.

Warnings and Precautions
• Because of the hydrochloric acid, metucellosis isolation is extremely important when using this product.
• Avoid contact with soft tissue. If product comes in contact with soft tissue, immediately rinse area thoroughly with water.
• Patient, clinician, and assistant must wear protective eyewear.

• Use only recommended tips.
• Keep away from heat and direct sunlight.
• If rubber dam is not used, follow instructions for OpalDam application.
• If allergic reaction is observed, rinse exposed area thoroughly and have the patient consult their physician.

Stepwise Instructions
Preparation
1) It is recommended to take initial photographs for reference.
2) Using rubber dam and clamps, or following instructions for OpalDam/OpalDam Green application, isolate tooth or teeth to be treated along with one tooth adjacent on each side.
3) Remove Luer lock cap from Opalustre syringe and securely attach White Mac™ tip.
4) Verify flow prior to applying intraoral. If resistance is met, replace tip and re-check.

Use
1) Apply Opalustre layer over the area to be treated.
2) Using an OpalCups Bristle at a slow RPM (approximately 500 RPM), apply intermittent medium to heavy pressure for approximately 60 seconds per application.
3) Remove the paste with suction then rinse, evaluate and repeat every 0.2mm has been removed, and repeat as necessary.

Cleanup
1) After final rinse, remove rubber dam, or OpalDam/ OpalDam Green thoroughly.
2) Use the OpalCups Finishing for surface polishing. A fluoride treatment is also recommended.

Contraindications
• Do not use on severe fluorosis.
• Do not use on thin enamel.
• For patients or users with allergy concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.

Warnings and Precautions
• Because of the hydrochloric acid, metucellosis isolation is extremely important when using this product.
• Avoid contact with soft tissue. If product comes in contact with soft tissue, immediately rinse area thoroughly with water.
• Patient, clinician, and assistant must wear protective eyewear.

• Use only recommended tips.
• Keep away from heat and direct sunlight.
• If rubber dam is not used, follow instructions for OpalDam application.
• If allergic reaction is observed, rinse exposed area thoroughly and have the patient consult their physician.

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