Ultrapro™ Tx

Description

Ultrapro Tx flavored prophy pastes are enamel-polishing agents with 1.23% fluoride ion. The flavored pastes are available in 3 grits and 4 flavors. Ultrapro Tx PURE is an enamel-polishing agent with no oils, fluoride, or dyes. PURE is flavor free and comes in medium grit. All pastes are packaged in 1.8 gram unit-dose cups.

Indications for Use

For use in dental polishing and cleaning procedures as part of a professional dental prophylaxis.

Note: Prophylaxis pastes containing fluoride are not considered a replacement for topical fluoride therapy.

Instructions

- Use with a prophy angle and rubber cup combination.
- Fill the cup by pressing the cup into the paste at a slight angle while running the handpiece at low speed.
- Apply paste to the tooth in quick, light strokes while slightly flaring the cup edge.
- Check and refill the cup as needed to avoid excessive friction to the tooth surface.
- Periodically rinse and suction or allow patient to rinse and expectorate.

Precautions and Warnings

- Carefully read and understand all instructions before using.
- For use by individuals who are professionally trained in coronal polishing.
- Do not use on patients with known allergies to fluoride. *Does not apply to PURE which is fluoride free.
- Excessive fluoride ingestion can cause nausea and/or vomiting. DO NOT SWALLOW. Rinse and suction or have patient expectorate. *Does not apply to PURE which is fluoride free.
- Ultrapro Tx Prophy Paste may cause eye irritation upon contact. Patient and clinician should wear protective eyewear when using this product. If eye contact occurs, immediately flush eye with copious amounts of water for 15 minutes. Seek medical attention if irritation persists.
- Clinician should wear protective gloves when using this product. Repeated skin contact may cause chronic dermatitis. If contact occurs, wash thoroughly with soap and water.
- To avoid splatter during treatment, do not exceed 45 psi when operating handpiece and prophy angle.
- Dispose of unit-dose cup after each procedure.
- For patients or users with allergy concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.

For additional information about using Ultrapro Tx Prophy Paste and other related products, please go to www.ultradent.com.

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