# Chromaclone®

Putty

**Description:**

Chromaclone Putty is a hydrophilic addition silicone (Polyvinylsiloxane) to be used for impressions and bite registrations that demand ultimate precision. It is pink in color after mixing and has a bubble gum smell and flavor.

**Indications:**

Chromaclone Putty is an excellent PVS base for final impressions for crown and bridge, implants, veneers, inlays, onlays, removable prosthodontics, etc. It is to be used as a first impression in the double impression technique and is followed by a wash. It can also be utilized by itself to make a preliminary impression to fabricate acrylic temporary crowns.

**Procedure:**

1. Coat the tray with Chromaclone Universal Tray Adhesive and allow tray to dry before placing material in tray.
2. Remove lids from base and catalyst tubs.
3. Use dosing spoons to obtain identical proportions of base and catalyst.
4. Immediately replace lids on base and catalyst tubs.
5. Mix identical proportions of base and catalyst with clean bare hands until an evenly colored mass is achieved with no streaks or color inconsistencies (Approximately 30 seconds).
6. Load mixed base/catalyst into tray.
7. Place in patients' mouth and allow it to set up completely (4-6 minutes).
8. When ready to make final impression, inject medium body, light body or super light body impression material into tray and around preparation and seat tray in mouth

Working Time            1:15 minutes

Setting Time            1:45 minutes

**Precautions/Warnings:**

1. Equal proportions must be used for proper polymerization.
2. If catalyst and base are massaged separately before mixing, the setting time of the mixed material will be accelerated.
3. Close tubs immediately after use to maintain freshness.
4. Silicone setting can be inhibited by latex gloves, sulfur and oils from hands. Always mix putty with clean glove free hands or wear vinyl gloves.
5. For best results use with Chromaclone Universal Tray Adhesive.
6. Do not use this material on patients with known allergy to silicones.

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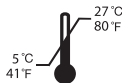
**Store at room temperature between 5°C - 27°C (41°F - 80°F).**

U.S. Federal law restricts this device to sale by or on the order of a state licensed dental professional.  
Keep out of reach of children.

For immediate order and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free: **1-800-552-5512**.  
Outside U.S. call (801) 572-4200,  
or visit [www.ultradent.com](http://www.ultradent.com).

© Copyright 2013 Ultradent Products, Inc. All Rights Reserved.

---

 Manufactured for Ultradent Products Inc.  
505 West 10200 South  
South Jordan, Utah 84095, USA  
Made in Italy

REF/UP 5040  
87480.7 060314

 **ULTRADENT**  
PRODUCTS, INC.  
  
UP Dental GmbH, Am Westhover  
Berg 30, 51149 Cologne Germany