Opal™ Etch

35% Phosphoric Acid

EN

Description:

Opal™ Etch is a 35% phosphoric acid etchant gel. The flowability allows precise placement. Opal Etch is self-limiting in its depth of etch (average depth of 1.5µ with 15 second etch), contains no glycerin, and is designed for rapid and complete removal upon rinsing.

Indications for Use:

Etching of enamel prior to bonding.

Directions for Use:

1. For 1.2ml syringe
- a. Remove Luer cap.
- b. Securely attach the Opal Etch tips.
- c. Verify etchant flow prior to applying intraorally.
2. Direct placement instructions
- a. Isolate tooth (teeth) to be etched.
- b. Pumice enamel.
- c. Rinse and dry prepared area.
- d. Verify etchant flow prior to applying intraorally.
- e. Apply etchant to enamel and leave in place 30 sec.
- f. Rinse thoroughly and dry. Enamel will appear chalky white.
- g. Continue with bonding procedure according to manufacturer's instructions.

Precautions and Warnings:

1. Carefully read and understand all instructions before using.
2. If product comes in contact with any soft tissue, IMMEDIATELY rinse area with copious amounts of water.
3. Replace cap, disinfect, and wipe syringe with an intermediate level disinfectant between uses.
4. Dispose of tip and empty syringe properly.
5. Verify flow of all syringes prior to applying intraorally. If resistance is felt, replace tip and re-check. Use only recommended tips.

NOTE: For SDS and additional information about using our products, please go to www.ultradent.com

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© Copyright 2022 Ultradent Products, Inc. All Rights Reserved.
Ultradent Products Inc.
Ultradent Products GmbH