# GISHY GOO!™

EN - Silicone Bracket Relief Aid

DE - Bracket Abdecksilikon

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**Description:**

Gishy Goo™ Silicone Bracket Relief Aid is a flavored, thixotropic, silicone elastomer that holds firmly to brackets and wires. It comes in a syringe with two barrels. One barrel has a white activator material. The other has a light yellow base material. A pea sized amount of Gishy Goo is dispensed by the patient into their hand, then mixed and applied to bracket or wires at the point of irritation. The product sets in two minutes and stays in place for up to 12 hours.

**Indications:**

Gishy Goo™ Silicone Bracket Relief Aid provides mechanical relief and a protective barrier to prevent irritation of soft tissue caused by orthodontic appliances. Gishy Goo™ Silicone Bracket Relief Aid is applied to points of irritation on brackets, wires and fixed/functional appliances.

**Ingredients:**

Metaloxide, Metalhydroxide, Polydimethylsiloxane, Paraffin, Silicates, Platinum Complex, Bubblegum Flavor, FD&C Yellow No 6.

**Storage:**

Keep Gishy Goo at room temperature. Do not store in hot places or direct sunlight, as Gishy Goo may not set effectively.

**General Warnings and Precautions:**

1. Warning: Do not use if you have an allergy to silicones.
2. Do not handle using latex gloves.
3. We highly recommend that children under the age of 12 be supervised or assisted by parents when using and applying Gishy Goo. Gishy Goo may represent a choking hazard to children 4 years of age and younger.
4. Do not eat. (Large amounts may present a choking hazard.)
5. The maximum amount of Gishy Goo for each sore spot is about the size of a pea. If you squeeze out too much, throw it away and try again.
6. You should be mixing equal amounts of white and colored Gishy Goo. If you don't have about the same amounts, throw away and try again.
7. Take Gishy Goo off of brackets before sleeping, eating, or drinking, to avoid the chance of choking.
8. Store Gishy Goo at room temperature.
9. Don't store Gishy Goo near heaters or leave it in hot cars, or it might not set properly.
10. Don't squeeze or stick Gishy Goo on clothes or fabric, as it may stain.

Remove cap by twisting to the left one-quarter turn (cap only twists one way) and pulling cap off.

Squeeze out a pea-sized amount onto finger or hard surface. Be sure to get the same amount of both white and colored material. If only one color comes out, throw that material away and try again.

Mix by hand for about 10 seconds.

Place mixed Gishy Goo on the brackets at the point of irritation. Gishy Goo takes about 2 minutes to set.

After use, put the cap back on and twist one-quarter turn to right. Make sure cap is on tightly.

Gishy Goo may remain in place for up to 12 hours. Remove all Gishy Goo from brackets before going to sleep, eating or drinking.

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© 2018 Ultradent Products, Inc. All rights reserved.
Ultradent Products GmbH