Gishy Goo™ — EN

Product Description

Gishy Goo Silicone Bracket Relief Aid is a flavored, thixotropic, silicone elastomer that holds firmly to brackets and wires. It comes in a syringe with two barrels. One barrel has a white activator material. The other has a light-yellow base material. A pea sized amount of Gishy Goo is dispensed by the patient into their hand, then mixed and applied to brackets or wires at the point of irritation. The product sets in two minutes in place for up to 12 hours. For all products described, carefully read and understand all instructions and SDS information prior to use.

Delivery Form(s)

- 5ml dual barrel syringe

Indications for Use/Intended Purpose

Gishy Goo Silicone Bracket Relief Aid provides mechanical relief and a protective barrier to prevent irritation of soft tissue caused by orthodontic appliances. Gishy Goo Silicone Bracket Relief Aid is applied to points of irritation on brackets, wires and fixed/functional appliances.

Do not use on patients with known allergies to silicones.

Warnings and Precautions

- Children under the age of 12 should be supervised or assisted by parents when using and applying Gishy Goo. Gishy Goo is a choking hazard to children 4 years of age and younger.
- Take Gishy Goo off brackets before sleeping, eating, or drinking, to avoid the chance of choking.
- Do not use. (Large amounts may present a choking hazard.)
- Do not handle using latex gloves.
- The maximum amount of Gishy Goo for each spot is about the size of a pea. If you apply too much, throw it away and try again.

Contraindications

Do not use on patients with known allergies to silicones.

Warnings and Precautions

- Children under the age of 12 should be supervised or assisted by parents when using and applying Gishy Goo. Gishy Goo is a choking hazard to children 4 years of age and younger.
- Take Gishy Goo off brackets before sleeping, eating, or drinking, to avoid the chance of choking.
- Do not use. (Large amounts may present a choking hazard.)
- Do not handle using latex gloves.
- The maximum amount of Gishy Goo for each spot is about the size of a pea. If you apply too much, throw it away and try again.

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