

THE



CLEAR

APPROACH TO HEMOSTASIS

- SPECIALLY DESIGNED for the esthetic zone
- TRANSPARENT GEL stays on the sulcus
- LEAVES NO RESIDUE and rinses off easily
- DOES NOT INTERFERE with bonding

ViscoStat® Clear  
25% aluminum chloride



ULTRADENT  
PRODUCTS, INC.

800.552.5512 [ULTRADENT.COM](http://ULTRADENT.COM)



## ACHIEVE HEMOSTASIS . . . ESTHETICALLY

This viscous yet spreadable 25% aluminum chloride gel quickly stops minor bleeding and sulcular fluid without the mess of a liquid or paste.



Subgingival preparation and bleeding sulcus.



Rub hemostatic firmly against bleeding tissues. Achieve the most rapid hemostasis with the Metal Dento-Infusor® tip. The clear gel allows easy visibility and rinses away quickly.



Press Ultrapak® cord into gel in sulcus around tooth preparations. Leave 4–5 minutes.



Remove cord. Firm air/water spray. Air dry. Rub gel into the sulcus again. Leave 1 minute.



Finished restoration 2 weeks post-op. Facilitates great control in the esthetic zone with no residue.



- **SPECIALLY DESIGNED** for the esthetic zone
- **TRANSPARENT GEL** stays on the sulcus
- **LEAVES NO RESIDUE** and rinses off easily
- **DOES NOT INTERFERE** with bonding\*

### 6407 ViscoStat Clear Dento-Infusor IndiSpense Kit

1 x 30 ml IndiSpense syringe  
20 x Metal Dento-Infusor tips  
20 x 1.2 ml empty syringes

### 6409 ViscoStat Clear Dento-Infusor Kit

4 x 1.2 ml syringes  
20 x Metal Dento-Infusor tips

### 6408 ViscoStat Clear IndiSpense Refill

1 x 30 ml IndiSpense syringe

### 6410 ViscoStat Clear Econo Refill

20 x 1.2 ml syringes

\*Data on file



Ultradent Products, Inc.

**TISSUE MANAGEMENT**

— Since 1978 —

**800.552.5512** ULTRADENT.COM

© 2019 Ultradent Products, Inc. All rights reserved.  
98035.15 101118

**ULTRADENT**  
PRODUCTS, INC.

Improving Oral Health Globally