

# How to Use Bleach Germicidal Cleaner Spray



**1. REMOVE**  
gross soil.



**2. SPRAY**  
6-8 inches from  
surface, until surface  
is thoroughly wet.



**3. WAIT**  
before wiping.  
Wait 1 minute  
for general  
disinfection,  
3 minutes for  
*C. difficile* spores.



**4. WIPE**  
surface with a  
clean damp cloth\*  
or paper towel to  
prevent residue  
build up.

**Clorox Healthcare®**  
**Bleach Germicidal Cleaner**  
Item # 68970, 32 fl. oz. Trigger Spray

\*Cloths can include:  
100% polyester synthetic dry  
wipes/ clean "microfiber",  
cotton cloths or paper towels

# Best Practices for Using Spray Disinfectants



- ▶ Spray disinfectant directly onto the surface.



- ▶ Apply enough disinfectant to ensure the surface stays wet for the contact time.



- ▶ Make sure the surface stays wet for the contact time listed on the label.



- ▶ Staff who might be sensitive to sprays can wear a mask.



- Refill Instructions:
- ▶ Remove trigger sprayer from empty bottle.
  - ▶ Unscrew cap on refill and pour contents directly into empty bottle.
  - ▶ Use a funnel to minimize product spills.
  - ▶ Replace trigger sprayer and use as directed.



HEALTHCARE™

FROM CLOROXPRO™

For product resources and implementation tools, contact your Clorox sales representative or Call: 1-800-234-7700  
Visit: [www.CloroxHealthcare.com](http://www.CloroxHealthcare.com)  
Email: [healthcare@clorox.com](mailto:healthcare@clorox.com)

© 2020 Clorox Professional Products Company,  
1221 Broadway, Oakland, CA 94612.  
NI-52472